


3! | PIL-MA

FLEX | HYDROPHILIC PRELOADED



3 CLICKS FOR
EASY, SAFE AND
PREDICTABLE INJECTION

MEDICENTUR

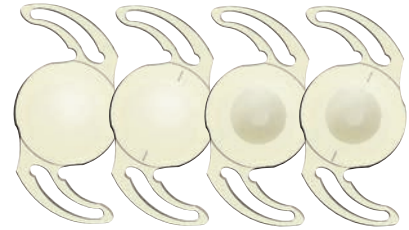

1st CLICK LOADING

Insert the injector to the wet container

2nd CLICK CLOSING

Fold up the cartridge nozzle


3rd CLICK READY

Push the red stopper forward to the end position and fill visco


INJECTION

Push the plunger slowly to the end position


Lens types	Single-piece aspheric IOLs, clear and yellow <ul style="list-style-type: none"> • Monofocal • Toric • Diffractive Progressive • Toric Diffractive Progressive (Patent pending)
Diameters	6.0 mm (optic) 13.0 mm (overall)
Angulation	0° – asymmetrical design
Material	Copolymer of hydrophobic and hydrophilic monomers, 25% water content with UV absorber / + blue light filter (390–470 nm)
PCO protection	360° Special Square Edge
A-Constant	118.9 (SRK/T)




677P/677PY
 Powers available -10.0 D → -1.0 D · (increment: 1.0 D)
 0.0 D → +30.0 D · (increment: 0.5 D)
 +31.0 D → +35.0 D · (increment: 1.0 D)



677PT/677PTY
 Powers available -10.0 D → -1.0 D · (increment: 1.0 D)
 0.0 D → +30.0 D · (increment: 0.5 D)
 +31.0 D → +35.0 D · (increment: 1.0 D)
 Cylinders available 1.00 D, 1.50 D, 2.25 D, 3.00 D, 3.75 D, 4.50 D



677PMY
 Powers available 0.0 D → +35.0 D · (increment: 0.5 D)
 Addition +3.5 D



677PMTY
 Powers available +5.0 D → +30.0 D · (increment: 0.5 D)
 Cylinders available 1.00 D, 1.50 D, 2.00 D, 2.50 D, 3.00 D, 3.50 D, 4.00 D, 4.50 D
 Addition +3.5 D